


In the ExoDx Prostate Test collection kit:

- ✓ Collection mailer with Fedex label
- ✓ Urine collection device
- ✓ Biohazard bag
- ✓ Test requisition form
- ✓ Cold packs
- ✓ Absorbent sheet
- ✓ Barcoded stickers


STEP 1

Complete test requisition form and fill in patient name and date of birth on the label. Print demographics or attach face sheet.

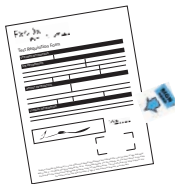

STEP 2


Set up urine collection device as shown, and collect sample.

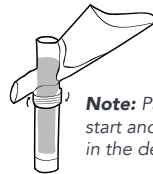

Note: Patient must start and finish void in the device.

STEP 3

Unscrew collection tube from funnel, close with provided lid and attach barcodes to tube and form.

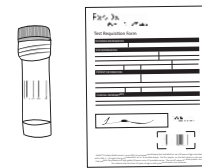

STEP 4

Place sample inside biohazard bag with the absorbent sheet. Place test requisition form and demographics or face sheet in the outer pouch.



STEP 5

Place biohazard bag in collection mailer with two (frozen) cold packs. Peel the backing from the sealing strip on the box flap and seal the mailer. **Notify FedEx for pickup.**



Important office instructions

- When kits arrive, make sure to remove the ice packs and freeze immediately for at least **12 hours**.
- Sample must be refrigerated within **one hour** of collection, and can be kept refrigerated for up to **seven days**.
- Demographics or face sheet may be included with the test requisition form.
- All information on the test requisition form must be completed. HCP signature is required.



- Patient should urinate with a full bladder (when a patient has already given a sample, he should wait at least one hour before giving another).
- Sample should be collected when patient first presents for his appointment, or at a followed-up scheduled appointment if a urine sample has already been provided.

This test was evaluated and its performance characteristics determined by Exosome Diagnostics, Inc. Exosome Diagnostics is certified under the Clinical Laboratory Improvement Amendments (CLIA) Act of 1988 as qualified to perform high-complexity clinical testing. CLIA number: 22D2093470

Exosome Diagnostics and ExoDx are registered U.S. trademarks of Exosome Diagnostics, Inc.

MKT 127_10_2019

